

# Assembly Instructions



The Voyager® Compact comes in two carry cases.

One bag contains the Voyager® Compact poles including:

- 6x Female round horizontal poles (4x bottom vertical poles and 2x horizontal poles)
- 4x Male round horizontal poles
- 4x Top verticals with straps
- 4x Corner pieces - 2 with brackets that pair up as shown (Please note that the 4 corner pieces colours may vary from picture shown)



The other bag contains:

- 1x Voyager® Compact fabric (rolled up)
- 1x Inflatable mattress
- 1x Foot pump

## Step 1 - Assembling the base

Unroll the Voyager® Compact fabric roughly where you want to put the Voyager® Compact in the room, checking that the door is facing out into the room.

Give yourself plenty of space to work round it.



Pull out the side pole pockets along the sides of the fabric and undo the velcro flaps at each end.

Take one female and two male horizontal pole.

Slot them together, making sure that the female is the middle of the three poles.



Slide the connected poles into the side pockets along each side of the fabric at the base, leaving equal amounts of pole sticking out at each end.



Take the corner pieces and position them together in pairs at each end of the Voyager® Compact fabric.

One with a bracket and one without.

Face the rubber feet at each end down onto the floor.



Slot all the corner pieces onto the ends of the long horizontal poles till they 'clunk' into position.



Lift the corner pieces up in the middle and slot the shorter piece under and into the bracket of the other.

Push down gently til flat and repeat at the other end.



Fasten up the Velcro flaps around the joined-up end bars.

The base of the Voyager® Compact is now assembled.



## Step 2 - Assembling the upright poles


Take the 4x bottom vertical poles and push them into the vertical corner slots.



Slot the 4x poles with straps into the upright poles with the straps facing inwards, towards the centre of the Voyager® Compact.

Make sure the silver camlock buckles are at the top of the poles.

Please note: the poles are designed to lean outwards at this stage.

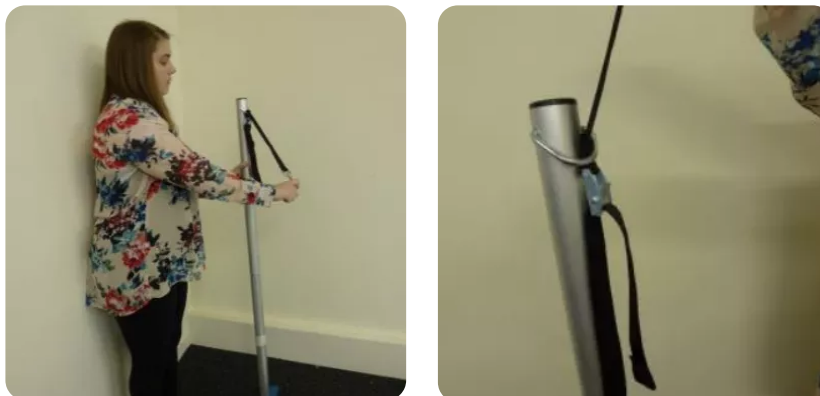

Making sure there are no twists on the strap, clip together the trigger hook and the D ring on the corner of the Voyager® Compact fabric so they are securely hooked together.

Repeat on all 4 corner.



Pull down on the straps about halfway - to pull the Voyager Compact but not fully at this stage.



## Step 3 - Fitting the mattress

Undo the floor zip and place the deflated mattress into the pocket.



Un-clip the hose from around the pump and screw it into the pump (hole marked 'out').

Put the nozzle onto the other end of the hose, put the nozzle into the mattress and pump up.  
Inflate the mattress and re-do the floor zip.





Pull down fully on the 4 pole straps so that the camlock buckles meet the bottom 'D' ring on the poles.

This will pull the Voyager Compact® Fabric up to the top and pull the poles in so that the whole structure is now within the boundaries of the base frame.

Do up the 2 rucksack style clips around all 4 corner poles.

It is nearly ready to use...



## **If you have a '5 piece mattress' please follow the instructions below -**

Assemble the 5 piece mattress outside of the compartment.

Once assembled slip one end of the mattress into the compartment and then push the mattress together at the same time as pushing it into the compartment.

## Step 5 - Final safety checks

Make sure the Voyager Compact is at least 30cm away from any walls or hard surfaces.

Test this by pushing the walls from the inside and making sure you cannot touch any walls, furniture or any hard objects.



## Step 6 - Using the Voyager® Compact



Use the Velcro dot on the roll-up door to hold the door open. Please see the Use & Care Guide for full instructions for use including Cleaning Advice. SECURING THE DOOR AT NIGHT. Please use your discretion and knowledge of your child regarding whether to clip the door closed for their own safety. Do not use a lock.

**Your Voyager® Compact is now ready to use!**

## Additional Safety Notice

Test the Voyager® Compact with your child or client under supervision regularly. Do this especially if there has been a long gap between use of The Voyager® Compact or your client has grown or their behaviour has changed.

## Dismantling the Voyager® Compact

To dismantle the Voyager® Compact work in exact reverse order

- Undo the rucksack corner clops around the poles
- Release the tension on the top corner straps
- Undo the floor pocket, remove and de-flate the mattress. Re-do the floor zip
- Release the top metal buckles
- Dismantle the upright poles
- Neatly fold the Voyager® Compact fabric so the net roof is flat as possible
- Dismantle and remove the corner pieces and base poles
- Roll up the fabric
- Pack it all away neatly into the two carry bags.

Make sure the Voyager Compact is clean and fully dried before dismantling and storing it

Over the years Safespaces have made changes in our designs. Please contact us if you have any difficulties.